

fold

DEVICE DESCRIPTION

The Affix CLIP™ is a sterile, single-use, stabilization device for indwelling lines/tubes/drainage catheters.

INDICATIONS FOR USE

The Affix CLIP™ is a device intended for medical purposes that consists of a strip of fabric material or plastic, coated on one side with an adhesive, and is used to secure objects to the skin.

CONTRAINDICATIONS

Size Restriction: Not to be used with any other Fr. size. Exclusively designed for the french size detailed on the label.

WARNINGS & PRECAUTIONS

- Do not use the device where loss of adherence could occur, such as on non-adherent, irritated, or compromised skin.
- Always follow the infection control guidelines at your facility.
- Minimize catheter or cable manipulation during application and removal of the device to prevent patient injury.
- The device and the surrounding area should be routinely inspected.
- Use sterile technique for application and removal of the device.
- In case of loss of performance or loosening of adherence, remove and replace immediately.
- Read all package insert warnings, precautions, and instructions prior to use.
- **Single Use Only:** Do not reuse, reprocess, or resterilize.
- Do not use if package is damaged or opened prior to use.

fold

INSTRUCTIONS FOR USE

Affix CLIP™ Catheter Securement

fold



fold


fold

A. INITIAL PLACEMENT

For patient comfort and safety, ensure the catheter is supported during removal.


1. Assemble the device.

Before placing on the body, align the grooved railing on dressing with indentation on clip (A), hold clip sideways at 90-degree angle and slide vertically into the dressing (B).

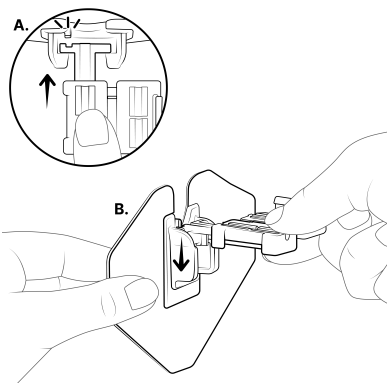


2. Position on the skin.

Remove the first peel off layer and place the "U" of the dressing onto the skin. Remove second and third layers once center is secure.

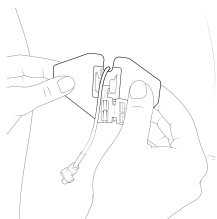

3. Secure the catheter.

Press the catheter into the lumen of the clip, and then close the clip around the catheter until it locks.

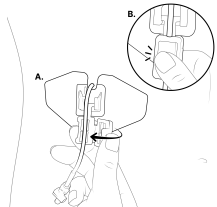


B. PATIENT / HOME CARE

For patient comfort and safety, ensure the catheter is supported during removal.


4. Remove the clip.

Turn the clip 90 degrees (sideways) to the dressing and slide it vertically upward to lift it out of the dressing.

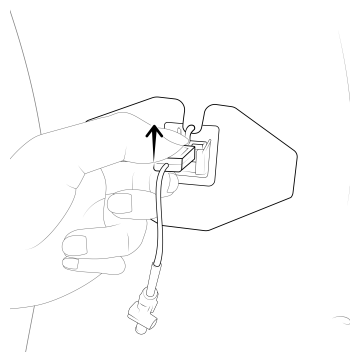

5. Remove the old dressing.

Slowly peel back the old dressing from your skin while carefully holding the clip and tube steady to prevent pulling.

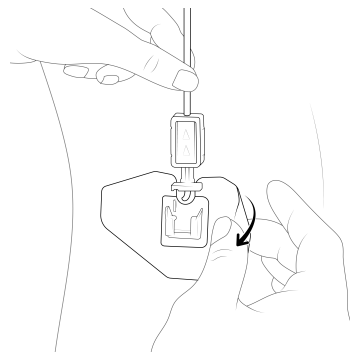


B. PATIENT / HOME CARE (CONT.)

For patient comfort and safety, ensure the catheter is supported during removal.


6. Apply the new dressing.

Apply the new dressing directly over the drain site as described in Step 2.

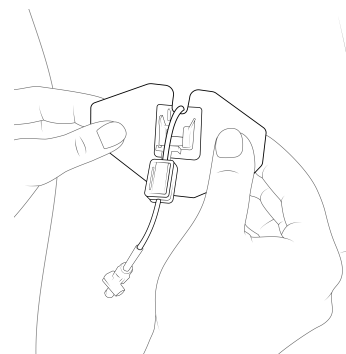

7. Reattach the clip.

Hold the clip at a 90-degree angle to the new dressing. Engage the rail on the dressing with the indentation on the clip (A), and slide the clip straight down to securely connect them (B).

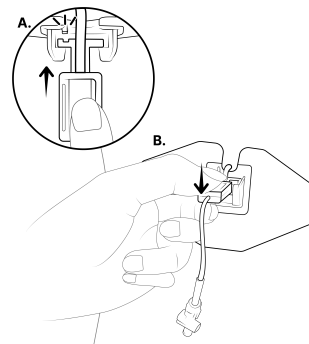

fold

GLOSSARY OF SYMBOLS (ISO 15223-1)

STERILE R	Sterilized using irradiation
	Single Use Only / Do not reuse
	Consult instructions for use
	Use By / Expiration date
LOT	Batch / Lot code
REF	Catalog / Reference number
	Temperature limits (See product label)
	Do not use if package is damaged
	Date of manufacture
	Manufacturer

fold

Affix Medical, Inc.

1623 W Fulton St, Chicago, IL 60612

© 2026 Affix Medical, Inc. All rights reserved.

Conforms to ISO 20417:2021